

KARIS FACIAL – QUICK MANUAL

1. Overview

KARISFACIAL is a professional skin and scalp treatment device that integrates nanotechnology, red laser toning (660 nm), microcurrent (less than 1 mA non-aggressive waves), negative-ion technology (989 cc), and galvanic infusion into one system. It is designed for estheticians, spas, and clinics as an all-in-one solution for cleansing, nourishing, and soothing the skin and scalp within a single treatment cycle.

2. Functions & Benefits

Nano Mist (Nanotechnology): Breaks down skincare solutions into nano-sized particles for deeper penetration, enhancing absorption for hydration and nourishment.

The **KARIS Stem-Cell Ampoule** is specially formulated with a thinner viscosity to create a fine nano mist without clogging the handpiece, and should be used exclusively for optimal results.

****Use of ampoules other than the KARIS Stem-Cell Ampoule may lead to damage of the White Nano Handpiece that cannot be restored through standard A/S repair.***

Red Laser Toning (660 nm): Stimulates cellular regeneration and circulation. Promotes collagen remodeling and skin healing.

Microcurrent (<1 mA, non-aggressive waves): Provides gentle electrical stimulation to lift, tighten, and tone facial muscles. Improves blood circulation and supports collagen and elastin production. Helps reduce fine lines and improves firmness.

Negative-Ion Technology (989 cc): Releases ions that detoxify, calm, and balance the skin. Reduces oxidative stress and supports skin clarity. Provides a soothing effect, especially post-infusion.

Galvanic Infusion (Electroporation): Uses safe electrical pulses to temporarily open micro-channels in the skin. Enables deep, needle-free infusion of active ingredients. Maximizes nutrient delivery for rejuvenation and elasticity.

Overall Benefits: Skin & scalp cleansing; Sebum, keratin & dandruff removal; Hydration, brightening & rejuvenation; Hair-loss prevention & scalp nourishment; Relief of itching, irritation & dermatitis; Lifting, toning, detoxifying & circulation enhancement.

3. Key Features

- Face & scalp all-in-one care
- Cleansing → Infusion → Soothing cycle
- Advanced technologies for hydration, regeneration & toning
- Built-in safety features ensure comfort and stable operation

4. Control Panel

- ① Start/Stop Nano Mist
- ② Start/Stop Red Laser (T.B.)
- ③ Start/Stop Microcurrent + Galvanic / M.E.S
- ④ TIME + ⑤ TIME –
- ⑥ LEVEL + ⑦ LEVEL –

5. Operation Basics

- 1) Fill White Nano Handpiece clear chamber halfway with KARIS Stem-Cell Ampoule only.
- 2) Connect handles (Nano Mist, Microcurrent/Galvanic, Ionto, Red Laser).
- 3) Power on device.
- 4) Select function → adjust time & intensity.
- 5) Start from M.E.S levels 1-5; adjust up to 10 as comfortable for the client, as it is safe.
- 6) Clean tips and handles after each session.

6. Precautions

- Avoid use on infants, children, or during pregnancy/lactation; safety not evaluated.
- Avoid use on individuals allergic to any ingredients; perform a patch test if needed.
- Do not aim the laser at eyes.
- Avoid eyelids, surgical sites, or diseased skin.
- Adjust time & intensity to client skin condition.
- Overuse may cause irritation.
- Not for children or those unable to express discomfort.
- Stop use if adverse reactions occur.
- Consult a physician if health conditions exist.
- Always review manuals before operating.

7. Specifications

Input Voltage: 110V / 50–60 Hz

Power: 100W

Main Unit: 610 × 440 × 400 mm | 7.5 kg

Cart: 680 × 470 × 400 mm | 17.5 kg

8. Warranty & Service

Warranty: 1 year (manufacturing defects only). Excludes misuse, impact, unauthorized repair, or wrong power supply.

Service Center: J & i Enterprises, Inc.

3000 W. 6th St., #206, Los Angeles, CA 90020

Email: info@aomedusa.com | Tel: 909.568.7996

Tip: Keep this Quick Manual with the device for safe, effective, and professional operation.