

Micro-needle RF Treatment Guideline

Treatment	Area	INT(in sec)	DEP(Depth)	RF level	EMI	Pass	Treatment Interval
Skin rejuvenation	Forehead	3	0.9	3	50	1	
	Eyes (periorbital)	3	0.5	3	50	1	
	Cheek	3	2.0 / 3.0	5	150	1	
	Zygomatic	3	1.3	4-5	150	1	
	Neck	3	0.7	4-5	150	1	
	Lip (peri-oral)	3	0.5	3	150	1	
	Nose	3	0.8	3	150	1	
Stretch mark	Body	3	0.6 / 0.8	5	250	2	4 - 6 weeks
Acre	Cheeks	3	0.5 / 1 / 1.5	4 / 5	200	1-2	
	Body (backs, arms)	3	0.5 / 0.8	5	250	1-2	
Pore	Cheek	3	0.5 / 0.8	4 / 5	100	1-2	
	Nose (ala area)	3	0.8 / 1.5	3 / 4	100	1-2	
	Glabella	3	0.5 / 0.8	3	100	1	
	Mentum	3	0.5 / 0.8	3	100	1	
	Cleavage	3	0.7	5	100	2	

*1 ~ 3 treatments is recommended to see effective result.

**Post treatment must be done with skin rejuvenative cream or cells.

**Based on clinical test for anti-aging purpose except for other indications of stretch mark and scars

Precaution during operation

- For the skin having wrinkles such as nasolabial fold and middle of forehead, must stretch wrinkles out as much as possible during procedure.
- There would be a line of Erythema or slight Edema in treated area depending on the patients. Symptoms are different depending on the patient but usually it disappears after 1~3 hours after treatment. In the case of severe erythema, calm the skin by Cryo, Cool pack.
- The skin should be properly contacted with the tip of hand-piece during procedure.

1) Treatment guide for Wrinkle



< Stretch the Wrinkles properly
for the exact needle injection >

2) Treatment Guide for Depth



< Needle Depth Guide
depending on the Area >